

## Prager Shell® Tray

Disconnect the tubing kit from Prager Shell® and discard the used kit. The Prager Shell® will not properly fit into the tray unless the kit is removed.

Fill the soaking tray approximately 2/3 with a fresh solution of 70% isopropanol alcohol or 3% hydrogen peroxide or other preferred aseptic solution.

Place the shell/probe into the storage/soak tray with the Luer fitting and setscrew facing the notched part of the tray opening. Let the shell/probe soak for at least five minutes. After removing the shell/probe from the tray, shake to dispel excess aseptic solution and let air dry. Flush the shell and probe with sterile saline before use. **Caution: aseptic solution is harmful to the cornea and must be completely rinsed from the shell and probe before patient use.**

Attach a new tubing kit. Do not keep the ultrasound probe soaking for a prolonged period a time.

**EIS, Inc.**  
**2915 Everest Lane N.**  
**Plymouth, MN 55447**  
**(763) 473-2533**